CE

Original Tape KINESIOLOGY

1. The roll of tape can be cut in various shapes and lengths required to fit for the size + shape of the muscles to be taped.

2. Also, wider tapes can be cut to the desired width before taping.

SHAPES OF TAPE

'I' shape

'Y' shape

'X' shape

'Finger' shape

STEP OF TAPING HEALING

Muscle pain

Taping Therapy

Pain-relief

Taping Tips

For correct use, consult with kinesiology taping expert

Do not stretch the tape when applying.

Test patch is recommended for those with high allergic tendencies.

After shower with tape on, do not rub water off but dry with clean towel.

Do not apply onto injured or irritated skin.

2-3days

After applying the tape for 2-3days, tape again after a day open .


Extend your muscles and apply tape without stretching the tape on the affected area. Start taping with the most painful area. Find the starting and ending points of the pain in the affected muscle and apply the tape with same direction as the muscle layers. Apply additional tape in the area with elevated level of pain.

Knee Taping

Shoulder Taping

Neck Taping

Elbow Taping

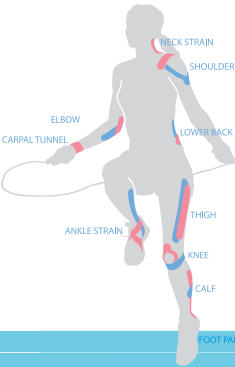Nasara Kinesiology Tape

Width 5cm x Length 5m

Nasara Kinesiology Tape

Width 5cm x Length 11m

Nasara Kinesiology Tape

Width 2.5cm x Length 5m

Nasara Kinesiology Tape

Width 7.5cm x Length 5m

Nasara Kinesiology Tape

Width 5cm x Length 32m

Nasara Kinesiology Tape

Quick Strips Pack

Nasara Kinesiology Tape

Quick Strips Bulk

Nasara Coporation

28, 139 beon-gil, Soraj-ro, Paju-si, Gyeonggi-do, Korea

www.nasara.co.kr

ENNOTES

Nasara Kinesiology Tape is applied on the affected muscles or joints of the skin and helps to relieve pain and promote body's natural healing process.

Nasara Kinesiology Tape can be used by individuals. Obtaining Knowledge of Kinesiology Taping Therapy and/or consultation with Taping Therapy Experts (doctors, technician, therapists, etc.) is recommended for optimal tape effectiveness and safety.

Please discontinue using the tape, and remove it with caution, if allergic reactions such as itching, redness, pain and blister occur. Please, see a doctor if necessary. To avoid such allergic reactions, small area patch test is recommended.

User precaution  
Not to stretch the tape while applying (especially both ends).  
After shower, please do not use a dryer on the tape but use towel to pat dry.

**Restriction**  
**Do not apply the tape if the skin is not intact or has open wounds (including bleeding).**  
**Do not use the tape on the skin with skin diseases and/or existing sores.**  
Please report to your local authority if any serious problem occurs.

- Non-sterile
- Single use only
- Keep away from the reach of children
- Store in cool area (10-30°C)
- No direct sunlight
- Materials-Cotton 96% Spandex 4%, Acrylic adhesive, water resistance

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MKROS GmbH